


# A6 Injector Set


Handling instructions for non-preloaded ZEISS MICS IOLs




Seeing beyond

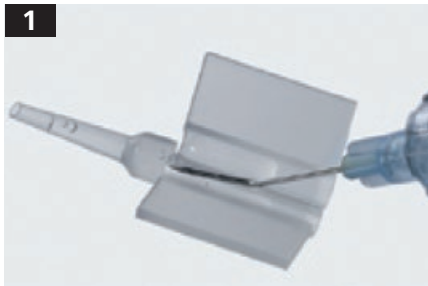

Fill the barrel halfway with viscoelastic substance. Put two lines of viscoelastics to the plane of the cartridge. Do not touch the tip of the cartridge.

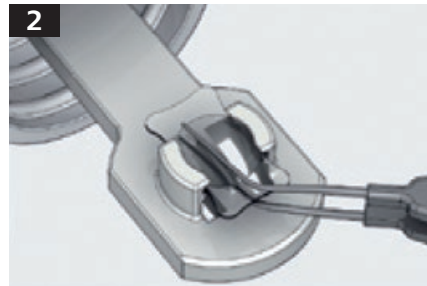

Pull the lens gently from its holder using a forceps with fine tips. Do not let the IOL dry up. Do not use the fluid from the bottle for ocular irrigation.

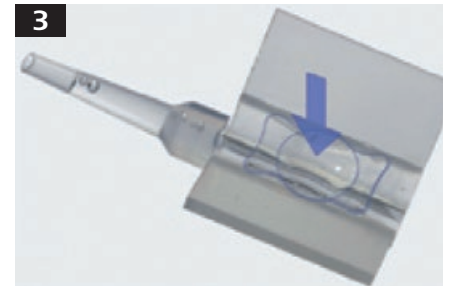

Place the lens in the center of the opened cartridge.



Hold optic down, center the IOL on the loading deck of the cartridge. Close the cartridge without clamping the lens.



Control the position of the lens within the channel of the closed cartridge. Haptic must move freely.



Injector A6 with groove for introduction of the Cartridge.



Push the loaded cartridge from the front into the groove until stop position.



Lock cartridge by holding the tip and rotating the injector.



Gently push plunger forward and check regular and continuous forward movement of the lens.

Please refer to [www.zeiss.com/injectors](http://www.zeiss.com/injectors) for diopter ranges applicable for each MICS IOL.



Cartridge



A6 Injector Set with silicone tip

**CE** 0120

A6 Injector Set



**MDJ SAS**

ZI Racine Palladuc  
63650 La Monnerie Le Montel  
France