



SKYJET Injector Set

Handling instructions for preloaded
CT ASPHINA 603P and CT SPHERIS 203P



Seeing beyond

SKYJET Injector Set

Preparing the preloaded CT ASPHINA® 603P and CT SPHERIS® 203P for use



1 Check that the plunger is fully withdrawn and held by the safety clips.



2 Insert the cassette into the injector following the arrows. Push all the way in until you hear a "click".



5 Hold the breakable safety tab between thumb and index finger. Snap it in the clockwise direction. The tab breaks and releases the lens.



7 Introduce viscoelastic into the hole on the back of the injector. Fill the first 3 mm of the injector nozzle up to the line.



8 Gently arm the plunger to release it from its safety clips and slowly push to drive the lens into the injector nozzle.



10 With the bevelled orifice facing downwards, insert the tip of the injector into the incision and inject the lens into the capsular bag.

CE 0459
SKYJET
CT ASPHINA 603P
CT SPHERIS 203P

 **Carl Zeiss Meditec AG**
Goeschwitzer Strasse 51–52
07745 Jena
Germany
www.zeiss.com/injectors

 **Carl Zeiss Meditec SAS**
27 Avenue Paul Langevin
17180 Périgny – France
Mailing address:
Avenue Paul Langevin – BP5
17053 La Rochelle – Cedex 9
France

EN_32_022_002711 Printed in Germany. C2-1/2020 International edition. Only for sale in selected countries.
The contents of the brochure may differ from the current status of approval of the product or service offering in your country. Please contact our regional representatives for more information. CT ASPHINA, CT SPHERIS and SKYJET are either trademarks or registered trademarks of Carl Zeiss Meditec AG or other companies of the ZEISS Group in Germany and / or other countries.
© Carl Zeiss Meditec AG, 2020. All rights reserved.