




**CT LUCIA 621P**  
Handling instructions




Seeing beyond

# Preparing the CT LUCIA 621P

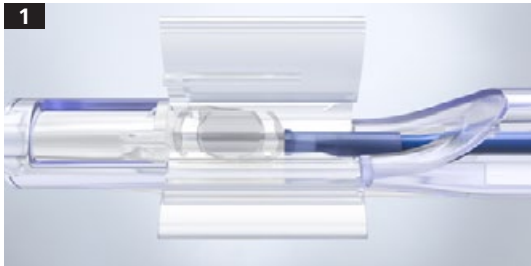


Verify that the lens is centered and secure in the IOL chamber.

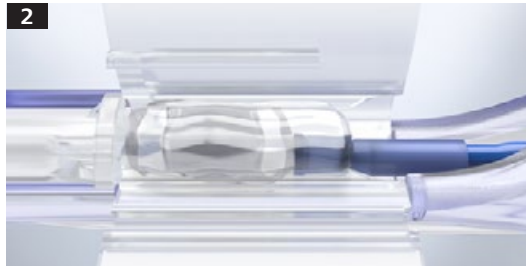

Cover the whole lens and blue plunger tip with a **generous** amount of ophthalmic viscoelastic. Avoid touching the lens and blue plunger tip.




Close the wings of the IOL chamber. **IMPORTANT:** Allow the lens to remain in this position until the surgeon is ready to implant it into the eye.

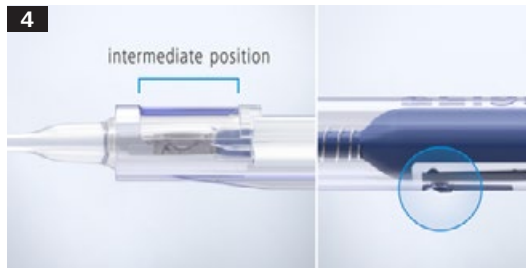

Advance the lens to the intermediate position. Gently push the plunger forward until an audible 'click' is heard.

**IMPORTANT:** The lens should be implanted immediately.

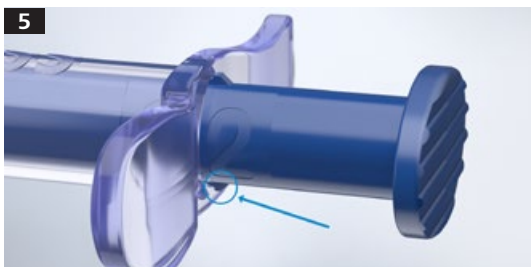

Before implantation, please check to ensure that the IOL is correctly oriented.

Injection: With loading chamber wings facing upwards, slowly advance the lens until it has been released from the injector.

If delivery is incomplete, apply additional force to the thumb flange to release the lens.

**6** Carefully position the lens in the capsular bag.

**7** Discard the device. Do not reuse the injector.

en-US\_32\_022\_016611 CZ-IV/2023 United States Edition: Only for sale in selected countries.  
The contents of the brochure may differ from the current status of approval of the product or service offering in your country. Please contact our regional representatives for more information.  
Subject to changes in design and scope of delivery and due to ongoing technical development. CT LUCIA is either a trademark or registered trademark of Carl Zeiss Meditec AG or other companies of the ZEISS Group in Germany and/or other countries.  
© Carl Zeiss Meditec USA, Inc., 2023. All rights reserved.

## Carl Zeiss Meditec USA, Inc.

5300 Central Parkway  
Dublin, CA 94568  
USA  
[www.zeiss.com/us/med](http://www.zeiss.com/us/med)



## Carl Zeiss Meditec Production, LLC.

1040 S. Vintage Avenue, Bldg. A  
Ontario, CA 91761-3631  
USA  
[www.zeiss.com/us/ct-lucia](http://www.zeiss.com/us/ct-lucia)  
[www.zeiss.com/med](http://www.zeiss.com/med)